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MIXTER ENDOSCOPIC CHOLANGIOGRAPHY SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Mixer Endoscopic Cholangiography Set consists of a radiopaque polyethylene catheter, a stainless steel needle stylet, and a three-way plastic stopcock.

INTENDED USE

The Mixer Endoscopic Cholangiography Set is intended for diagnostic evaluation of the bile ducts during percutaneous cholecystectomy procedures.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

This product is intended for use by physicians trained and experienced in endoscopic cholangiography techniques. Standard techniques should be employed.

INSTRUCTIONS FOR USE

1. Make a small incision in the skin at the catheter insertion site, as shown in **Fig. 1**. **NOTE:** The insertion site is close to the costal margin and quite lateral. This allows the catheter to lie parallel to the cystic duct. The catheter should not be placed down a pre-existing trocar site.
2. Introduce the catheter/needle stylet assembly into the abdomen.
3. Withdraw the tip of the needle stylet approximately 1 cm into the catheter.
4. Place tension on the cystic duct by grasping the apex and neck of the gallbladder, then use scissors through the free port to make an incision in the duct.
5. Thread the catheter into the cystic duct.
6. Once the catheter is introduced into the cystic duct, remove the needle stylet.
7. Apply a clip to hold the catheter in place and seal the duct around the catheter. **NOTE:** Proximal clip placement may reduce the possibility of the catheter becoming dislodged.
8. Cholangiography may now be performed.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.