COOK®**MEDICAL****CE**
0123

EN 2	Cytology Brush
CZ 2	Cytologický kartáček
DA 3	Cytologibørste
NL 4	Cytologieborstel
FR 5	Brosse de cytologie
DE 6	Zytologiebürste
EL 7	Ψήκτρα κυτταρολογίας
HU 8	Citológiai kefe
IT 8	Spazzolino citologico
NO 9	Cytologibørste
PL 10	Szczoteczka cytologiczna
PT 11	Escova de citologia
ES 12	Cepillo de citología
SV 13	Cytologiborste

* 1 0 5 1 3 / 0 6 1 9 *

INTENDED USE

This device is used to collect cells from the upper and lower gastrointestinal tracts.

NOTES

Do not use this device for any purpose other than stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

WARNINGS

This device is designed for single use only. Attempts to reprocess, resterilize and/or reuse may lead to device failure and/or transmission of disease.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

CONTRAINDICATIONS

Those specific to the primary endoscopic procedure to be performed in gaining access to desired cytology site.

POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTION

Refer to package label for minimum channel size required for this device.

INSTRUCTIONS FOR USE

1. Uncoil catheter and prepare for use.
2. Remove protective sheath with label from the brush and extend/retract brush.
3. Endoscopically inspect area to be sampled.
4. With cytology brush retracted, insert device into accessory channel of endoscope.
5. Advance in short increments until tip of device is visualized exiting endoscope.
6. Advance device to desired sampling site, then advance brush out of sheath by pushing forward on handle.
7. Brush area to be sampled several times to obtain adequate cellular material.
8. Pull brush back into sheath and withdraw device from endoscope.
9. Prepare specimen for examination per institutional guidelines.

Upon completion of procedure, dispose of device(s) per institutional guidelines for biohazardous medical waste.

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© 2019 Cook Medical
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.