EN 2	Memory™ Polyp Retrieval Baskets
CS 3	Košíky Memory™ k extrakci polypů
DA 4	Memory™ polypudtagningskurve
NL 5	Memory™ poliepvangbaskets
FR 6	Paniers Memory™ pour ablation de polypes
DE 7	Memory™ Körbchen für die Polypenentfernung
EL 9	Καλάθια ανάκτησης πολύποδα Memory™
HU 10	Memory™ polipeltávolító kosarak
IT 11	Cestelli per il recupero di polipi Memory™
NO 12	Memory™-polypputtrekkingskurver
PL 14	Koszyki Memory™ do wydobywania polipów
PT 15	Cestos de recuperação de pólipos Memory™
ES 16	Cestas para extracción de pólipos Memory™
SV 17	Memory™ polypfångarkorgar

INTENDED USE

This device is used for the endoscopic removal of polyps after polypectomy.

NOTES

Do not use this device for any purpose other than stated intended use.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Contraindications include those specific to primary endoscopic procedure to be performed in conjunction with removal of polyps.

POTENTIAL COMPLICATIONS

In addition to those associated with gastrointestinal endoscopy procedures, complications associated with retrieval of a polyp after polypectomy include, but are not limited to: tissue irritation.

PRECAUTIONS

Refer to package label for minimum channel size required for this device.

This device should never be coiled in less than an 8-inch (20 cm) diameter.

WARNINGS

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

This device is supplied non-sterile, and therefore should not be used in a sterile field.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

INSTRUCTIONS FOR USE

1. With the basket fully retracted into the sheath, insert the device into an endoscope accessory channel. Advance the device through the channel in short increments until the basket sheath exits the endoscope.
2. Endoscopically visualize the polyp to be removed and advance the basket to the appropriate position.
3. Place the sheath of the basket beyond the target, and slowly extend the basket to entrap polyp. **Note:** It may be necessary to move the basket back and forth along the target in order to capture the polyp.
4. Withdraw the basket into the sheath while maintaining entrapment. Support the polyp against the endoscope tip, and withdraw the endoscope from the patient. Maintain endoscopic visualization to ensure continued entrapment of the polyp. While withdrawing device from endoscope, wipe secretions from outer sheath.
5. Prepare extracted specimen per institutional guidelines.

Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

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© 2019 Cook Medical
Wilson-Cook Medical, Inc.
Cook Medical Europe Ltd.