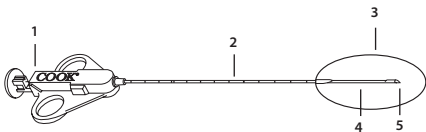
- EN**  
3 **Quick-Core®** Biopsy Needles and Sets  
Instructions for Use
- CS**  
3 **Bioptické jehly a soupravy Quick-Core®**  
Návod k použití
- DA**  
4 **Quick-Core®** biopsinåle og -sæt  
Brugsanvisning
- DE**  
5 **Quick-Core®** Biopsiekanülen und Sets  
Gebrauchsanweisung
- EL**  
6 **Βελόνες και σετ βιοψίας Quick-Core®**  
Οδηγίες χρήσης
- ES**  
7 **Agujas de biopsia y equipos Quick-Core®**  
Instrucciones de uso
- FR**  
8 **Aiguilles et sets de biopsie Quick-Core®**  
Mode d'emploi
- HU**  
8 **Quick-Core®** biopsziás tűk és készletek  
Használati utasítás
- IT**  
9 **Aghi e set per biopsia Quick-Core®**  
Istruzioni per l'uso
- NL**  
10 **Quick-Core®** biopsienaalden en -sets  
Gebruiksaanwijzing
- NO**  
11 **Quick-Core®** biopsinåler og -sett  
Bruksanvisning
- PL**  
12 **Igły i zestawy biopsyjne Quick-Core®**  
Instrukcja użycia
- PT**  
13 **Agulhas e conjuntos de biopsia Quick-Core®**  
Instruções de utilização
- SV**  
14 **Quick-Core®** biopsinålar och -set  
Bruksanvisning

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## QUICK-CORE® BIOPSY NEEDLES AND SETS

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

### DEVICE DESCRIPTION

The Quick-Core is a lightweight biopsy needle with a beveled point stylet and an etched tip. The Set includes a coaxial needle which can be placed prior to the Quick-Core, allowing access to a target lesion to take multiple samples with only one puncture.

### INTENDED USE

Quick-Core Biopsy Needles are intended for soft tissue biopsy. The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for soft tissue biopsy should be employed.

### CONTRAINDICATIONS

None known

### WARNINGS

None known

### PRECAUTIONS

- **When test firing Quick-Core Biopsy Needle, place distal beveled tip of inner stylet against solid surface before activating cutting cannula. This will prevent damage to cutting cannula tip.**
- Lung puncture may result in an air embolus, which could lead to ischemia or infarction of major organs, including the brain or cardiac system.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

### INSTRUCTIONS FOR USE

1. If using the coaxial needle, insert the coaxial needle to the border of the lesion.
2. Prior to insertion, prepare the Quick-Core Biopsy Needle by pulling back on the plunger until a firm click is felt. This indicates that the needle spring is locked into ready position.
3. With the stylet fully retracted, so that the specimen notch is completely covered by the cutting cannula, advance the needle to the area to be biopsied. **Do not advance stylet until Quick-Core Biopsy Needle is in position.**
4. While maintaining needle position, advance stylet with thumb, to expose specimen notch within the area to be biopsied.
5. Fire the cutting cannula by fully depressing the plunger with thumb to capture tissue within the specimen notch.
6. Withdraw the needle from the biopsy area.
7. To remove tissue specimen, pull back on the plunger until a firm click is felt. This indicates that the cutting cannula is locked into position. Push stylet forward to expose tissue specimen within the notch, and remove tissue.

### HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

### REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd