|                 |  |
|-----------------|--|
| <b>EN</b><br>5  | <b>Esophageal • Pyloric • Colonic Balloon Dilators and Inflation Device</b>                  |
| <b>CS</b><br>7  | <b>Jícnové • pylorické • kolonické balónkové dilatátory a plnicí zařízení</b>                |
| <b>DA</b><br>9  | <b>Ballondilatatorer og inflationspistol til øsophagus • pylorus • colon</b>                 |
| <b>NL</b><br>12 | <b>Oesofagus-, pylorus- en colonballondilatators en inflatie-instrument</b>                  |
| <b>FR</b><br>14 | <b>Dilatateurs à ballonnet et dispositif de gonflage œsophagiens • pyloriques • coliques</b> |
| <b>DE</b><br>17 | <b>Ballondilatatoren und Inflationsgerät für Ösophagus • Pylorus • Kolon</b>                 |
| <b>EL</b><br>20 | <b>Διαστολείς με μπαλόνι Οισοφαγικοί • Πυλωρικοί • Παχέος εντέρου και συσκευή πλήρωσης</b>   |
| <b>HU</b><br>22 | <b>Oesophagealis • pyloricus • colon tágitóballonok és feltöltőeszköz</b>                    |
| <b>IT</b><br>25 | <b>Dilatatori a palloncino esofageo, pilorico, colico e dispositivo di gonfiaggio</b>        |
| <b>NO</b><br>28 | <b>Ballongdilatorer og inflasjonsanordninger for øsofa-gus • pyloris • kolon</b>             |
| <b>PL</b><br>30 | <b>Poszerzadła balonowe przełyku • odźwiernika • okrężnicy i urządzenie do napełniania</b>   |
| <b>PT</b><br>33 | <b>Dilatadores com balão esofágicos • pilóricos • do cólon e dispositivo de enchimento</b>   |
| <b>ES</b><br>35 | <b>Balones de dilatación esofágica • pilórica • colónica y dispositivo de hinchado</b>       |
| <b>SV</b><br>38 | <b>Ballongdilatorer och fyllningsanordning för matstrupe • nedre magmun • tjocktarm</b>      |


0123

ECL, QD,  
QDC, QDP


QID

\* 1 5 5 9 8 / 0 7 1 9 \*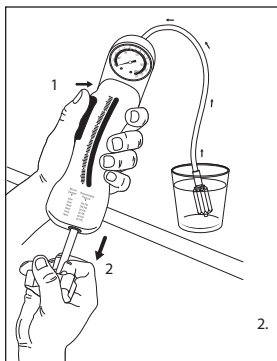


2.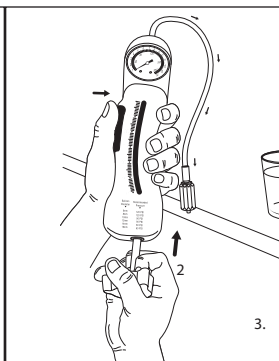


3.





- 1 I / D button  
 Tlačítko I/D  
 I/D-knap  
 Vullen/legen-knop  
 Bouton G/D  
 I/D-Taste  
 Κουμπί πλήρωσης/ξεφουσκώματος  
 Feltöltő/leeresztő gomb  
 Pulsante I/D  
 I/D-knapp  
 Przycisk napełniania/oprózniania  
 Botão E/E  
 Botón de hinchado/deshinchado  
 F/T-knapp

- 2 P / D knob  
 Knoflík P/D  
 P/D-knap  
 P/D-knop  
 Poignée P/D  
 P/D-Knauf  
 Λαβή P/D  
 P/D fogantyú  
 Manopola P/D  
 P/D-knapp  
 Gałka P/D  
 Botão P/D  
 Mando P/D  
 P/D-handtag

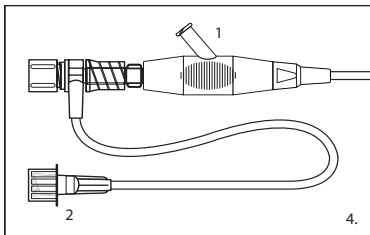- |   |  |
|---|--|
| <p>1</p> <p>Sideport<br/>Postranní port<br/>Sideport<br/>Zijpoort<br/>Orifice latéral<br/>Seitenöffnung<br/>Πλευρική θύρα<br/>Oldalnyílás<br/>Raccordo laterale<br/>Sideport<br/>Port boczny<br/>Orificio lateral<br/>Orificio lateral<br/>Sidoport</p> | <p>2</p> <p>Flush Port for Wire Guide Lumen<br/>Proplachovací port pro lumen vodícího drátu<br/>Skylleport til kateterlederlumen<br/>Spoelpoort voor voerdraadlumen<br/>Orifice de rinçage de la lumière du guide<br/>Spülanschluss für Führungsdrahtlumen<br/>Θύρα έκπλυσης για αυλό συρμάτινου οδηγού<br/>A vezetődrot befogadására szolgáló lumen öblítőnyílása<br/>Raccordo per il lavaggio del lume per la guida<br/>Skylleport for ledevaierens lumen<br/>Port do przepłukiwania kanału dla przewodnika<br/>Orificio de irrigação para o lúmen do fio guia<br/>Orificio de lavado de la luz de la guía<br/>Spolport för ledarlumen</p> |
|---|--|
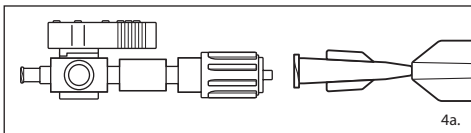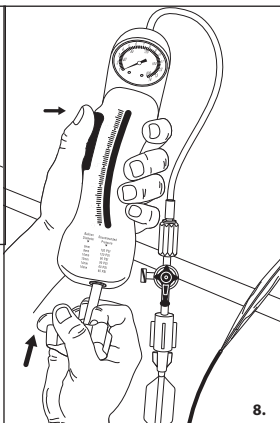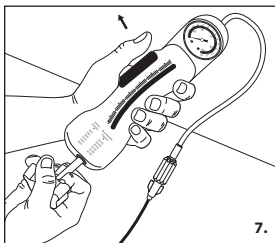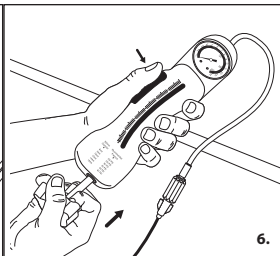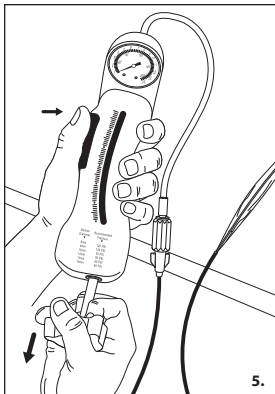
### INTENDED USE

This device is used to dilate strictures of the gastrointestinal tract, including strictures of the esophagus, pylorus, duodenum and colon.

The Q.I.D. - Quantum Inflation Device is used for inflation, deflation and pressure monitoring of the balloon dilator.

### NOTES

This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

Do not use this device for any purpose other than stated intended use.

If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

These devices are supplied non-sterile and may not be sterilized by any method.

### CONTRAINDICATIONS

Those specific to the primary endoscopic procedure to be performed in gaining access to the desired dilation site.

Those specific to dilation include, but are not limited to: uncooperative patient; asymptomatic rings, webs, or strictures; inability to advance balloon dilator through strictured area; coagulopathy; known or suspected perforation; severe inflammation or scarring near dilation site.

### POTENTIAL COMPLICATIONS

Those associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

### PRECAUTIONS

Refer to package label for minimum channel size required for this device.

Do not pre-inflate balloon.

### **Prior to insertion of balloon dilator, negative pressure is mandatory to maintain balloon deflation.**

Endoscope should remain as straight as possible when advancing or withdrawing balloon dilator.

Entire balloon should be extended beyond tip of endoscope, and be completely visualized and positioned, before inflation.

This device is used in conjunction with the Q.I.D. or manometer and fluid filled syringe. Do not use air or gaseous substances to inflate balloon, as this will result in reduced balloon effectiveness.

During withdrawal of balloon dilator from endoscope, negative pressure is mandatory to maintain balloon deflation.

### **Apply a water soluble lubricant to balloon to allow easier passage through accessory channel.**

### WARNINGS

Recommended 100% balloon inflation pressure can be found on Q.I.D. and on catheter tag of balloon dilator.

Do not expose balloon to temperatures in excess of 50°C/125°F.

Do not advance balloon dilator if resistance is encountered. Assess cause of resistance to determine if dilation should be reattempted.

## SYSTEM PREPARATION

1. Examine features of Q.I.D. (See fig 1). Ensure pressurize/depressurize (P/D) knob is rotated completely counterclockwise and plunger is pushed completely forward. To move plunger, depress and hold inflate/deflate (I/D) button. Release I/D button to lock plunger in position.
2. Submerge connecting tube in a minimum of 60 ml of water or saline.
3. Fill inflation device with a minimum of 40 ml and a maximum of 50 ml of fluid by depressing I/D button while simultaneously grasping and pulling back on P/D knob. (See fig. 2) Release I/D button to lock plunger in place.
4. To completely purge inflation device of air, hold Q.I.D. upright and depress I/D button while grasping and slowly pushing forward on P/D knob. (See fig. 3) Release I/D button to lock plunger in position.
5. Attach male Luer lock fitting of Q.I.D. connecting tube to sideport of y-connecting cap of balloon dilator for Eclipse (See fig. 4) or to hub of balloon dilator for Quantum. (See fig. 4a) Secure by rotating fittings. **Note:** If use of stopcock is desired, attach appropriate fittings to balloon dilator and Q.I.D. connecting tube. To secure attachment, rotate fittings. Arm of stopcock indicates which port is closed.
6. Aspirate all residual air from balloon.
7. Depress I/D button and pull back P/D knob until vacuum is indicated on pressure gauge. While maintaining a vacuum with the P/D knob, release I/D button. Release P/D knob. Balloon dilator is now ready for placement through accessory channel of endoscope.  
**Note:** Maintain balloon deflation with negative pressure. (See fig. 5)

## INSTRUCTIONS FOR USE

### Introduction

1. Remove protective sheath from balloon.
2. **Apply a water soluble lubricant to balloon to allow easier passage through accessory channel.**
3. Introduce deflated balloon into accessory channel and advance in short increments until dilator is completely visualized endoscopically.
4. Using fluoroscopic or endoscopic monitoring, continue advancement until desired position is achieved.
5. Before inflation, ensure balloon fully exits endoscope and is in desired position within stricture. **Note (if applicable):** To use with a wire guide, remove stylet and flush catheter lumen. Insert wire guide.

### Balloon Inflation

**Note:** Balloon must be inflated with **liquid only**. Use of air or gaseous substances to inflate balloon will result in reduced balloon effectiveness.

6. For rapid inflation, depress I/D button and push P/D knob forward until significant resistance is met. (See fig. 6) Maintain pressure on P/D knob while releasing I/D button. (See fig. 7)
7. To attain specified balloon pressure for dilator being used, rotate P/D knob clockwise. Refer to catheter tag of balloon for recommended pressure. Pressure may be decreased by rotating P/D knob counterclockwise.

### Balloon Deflation

8. To decrease balloon pressure and deflate balloon, hold inflation device while rotating P/D knob counterclockwise until pressure decreases below 20 psi. Depress I/D button and pull back on P/D knob, creating negative pressure as seen on pressure gauge. Release I/D button to lock in negative pressure.
9. Maintain negative pressure and observe balloon for complete deflation. Negative pressure may be increased by continued counterclockwise rotation of P/D knob. **Note:** Indicator on pressure gauge should register in red, vacuum portion of gauge during deflation.
10. If balloon is not completely deflated, disconnect Q.I.D. connecting tube from balloon and purge inflation device of air. To purge inflation device of air, hold upright and depress I/D

button while grasping and slowly pushing forward on P/D knob. **Note:** If optional stopcock is attached, open stopcock to atmosphere and purge inflation device of air by closing port to balloon catheter. Depress I/D button and push forward on P/D knob. (See fig. 8)

11. Reattach fittings and create negative pressure by depressing I/D button and pulling back on P/D knob. Repeat as necessary until balloon is completely deflated. **Note:** If stopcock is attached, turn stopcock to close open port and the create negative pressure.

### Balloon Withdrawal

**Caution:** A compromised balloon may prohibit removal from endoscope accessory channel. Removal of endoscope along with compromised balloon may be required.

12. To remove deflated balloon from accessory channel, straighten endoscope tip and withdraw balloon using a continuous twisting motion. **Caution:** Balloon must be completely deflated with all fluid removed before withdrawal. **Note (if applicable):** Previously positioned wire guide may be left in position to facilitate introduction of other wire guided devices.

**Upon completion of procedure, dispose of device per institutional guidelines for biohazardous medical waste.**

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© 2019 Cook Medical
**Wilson-Cook Medical, Inc.**
**Cook Medical Europe Ltd.**