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CE
0123

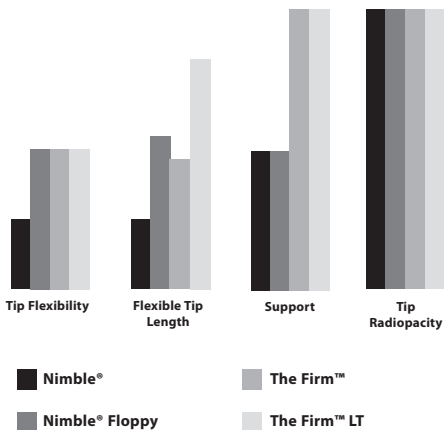
T _ R R P C _ R E V 3

NIMBLE®, NIMBLE® FLOPPY, THE FIRM™, THE FIRM™ LT ROADRUNNER® PC WIRE GUIDES WITH AQ® HYDROPHILIC COATING

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Roadrunner PC Wire Guide is coated with AQ (a biocompatible hydrophilic coating), which when wet becomes very lubricous, greatly reducing the coefficient of friction of the surface of the wire guide. The wire guide is made of a nitinol core with distal platinum inner spring coil tip and an outer polyurethane sleeve.

INTENDED USE

Roadrunner PC wire guides are intended for complex diagnostic and interventional procedures. The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of vascular access sheaths, angiographic catheters and wire guides should be employed.

CONTRAINDICATIONS

None known

WARNINGS

- Not intended for coronary use.
- Never use a Roadrunner PC wire guide through a metal needle cannula.** Destruction of the outer polyurethane coating may result. A plastic outer sheath needle is recommended if the Roadrunner PC wire guide is used for initial placement.

PRECAUTIONS

- Always keep the surface of the wire guide wetted for optimal results.**
- Manipulation of the product requires fluoroscopic control.
- Roadrunner PC wire guides are very slippery when wet. Always maintain control of the wire guide when manipulating it through the catheter.
- When the wire guide is removed from a catheter, wipe it with a sterile gauze pad moistened with heparinized saline solution or sterile water and place it back into the holder. Fill the holder with heparinized saline solution. This will maintain the lubricous state of its surface for subsequent procedural use.

INSTRUCTIONS FOR USE

Activating the AQ® Hydrophilic Coating

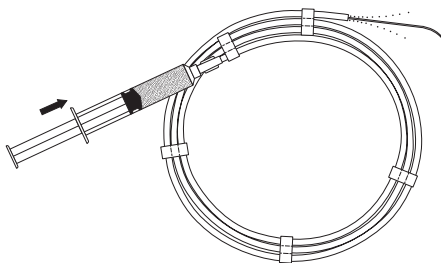
Fig. 1

1. Attach the syringe with heparinized saline solution or sterile water to the Luer lock fitting of the wire guide holder.
2. Inject enough solution to wet the wire guide surface entirely. (**Fig. 1**) This will activate the AQ coating, making the wire guide surface very lubricous.
3. Standard wire guide use may now be initiated. An initial flush of heparinized saline through the catheter lumen will enhance initial movement of the wire guide through the catheter.

NOTE: If the surface of the wire guide becomes dry after removal from the holder, wetting with additional heparinized saline or sterile water will renew the hydrophilic effect.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd